PACK TYPE	MANUFACTURING SITE	LINE
LEAFLET	ANCONA	THERMACARE

ⓑ Intended purpose:

Effective and targeted relief of knee pain associated with:

- overexertion
- strains and sprains
- arthritis
- muscle tension.

Suitable for those suffering from acute, chronic and occasional pain. The product is intended for self-sufficient patient population above 12 years of age.

Refer to warnings for limitations of use. For knee pain. 8 hours of constant heat. Therapeutic heat that penetrates deeply, for a prolonged relief.

Specially designed to fit your knee. Effective pain relief. Clinically proven to relax tight muscles and reduce knee stiffness. Proven to enhance flexibility and mobility.

Free from active pharmaceutical ingredients.

ThermaCare® heatwraps provide prolonged heat at constant temperature.

This increases blood circulation in the tissues, producing pain-relieving effects and relaxing the muscles, by:

- bringing more oxygen to damaged tissues, helping the muscle to return to a state of well-being.
- stimulating the nerve endings, sensitive to temperature, interrupting the transmission of pain and thus promoting muscle relaxation.

ThermaCare® heatwraps are made of soft and comfortable material with special patented cells that contain natural ingredients (iron, coal, salt and water) which develop heat upon contact with oxygen in the air.

Therefore ThermaCare®:

- Contains no drugs
- It is easy to use
- It is comfortable to wear, thin, discreet, odour-free.

In case of any serious incident that occurs in relation to the product, report to the Manufacturer, the Authorised Representative and to the local Authority.

Instruction for use:

- Tear open pouch when ready to use.
- Do not cut the wrap.
- Gently tear along the perforated notch to separate the lateral tails.
- Place over pain area with darker heat calls toward skin and fasten, ensuring the circular opening is over the kneecap.
- Adjust the tension of the individual tails so that the hook attaches to the landing zone. Refer to **Do not use** section for areas where not to apply.

- It may take up to 30 minutes for ThermaCare® to reach its therapeutic temperature.
- Remove the wrap after 8 hours.

The duration of treatment should not exceed 7 consecutive days. If 55 and older, refer to section **Special warnings and precautions for use**.

This product does not require any special storage conditions.

Special warnings and precautions for use

This product can cause burns/blisters. Check skin frequently during use. If you find irritation or a burn, remove product immediately.

For users that are not self-sufficient, or for users aged 12-18: supervision by a caregiver is required. 55 or older: Your risk of burning increases as you age.

If you are 55 years of age or older, wear ThermaCare® over a layer of clothing or cloth, and do not wear while sleeping.

Consult your healthcare professional before use if you have diabetes, poor circulation, heart disease, rheumatoid arthritis, or are pregnant.

Stop use and consult a doctor if you experience any discomfort, burning, swelling, rash, or other changes in your skin that persist where the wrap is worn; if after 7 days your pain gets worse or remains unchanged.

Each heat cell contains iron (~2 grams):

- If ingested please seek professional assistance immediately.
- If it comes in contact with your skin or eyes, remove wrap, rinse affected area with water and seek professional assistance immediately.
- If the wrap is damaged or torn during use, remove the product.

Keep out of reach of children and pets.

Never heat product in a microwave or attempt to reheat, as wrap could catch fire.

Do not use:

- If heat cell contents leak and/or wrap is damaged or torn or if the packaging is damaged
- With topical preparations
- On unhealthy, damaged or broken skin
- On areas of bruising or swelling that have occurred in the last 48 hours
- On people unable to follow all use instructions or unable to remove the product
- With other forms of heat
- Under 12 years of age
- On areas of the body different from the knee.

When using this product:

Do not use more than 8 hours in a 24-hour period.

Do not apply wrap too tightly and do not overlap the heat cells.

Do not place extra pressure over the product such as leaning against hard surfaces or wearing under tight clothing.

If product feels too hot, stop use or wear over a layer of clothing, not directly against your skin.

To avoid increased risk of burns, remove product before undergoing any MRI (magnetic resonance imaging) procedures.

Keep the heatwrap dry.

Consult the section **Instruction for use**.

At the end of use, dispose the product once it has exhausted its heating function, in accordance with local regulations of domestic waste collection, avoiding the dispersion of its contents in the environment.

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STANDARD INFO PANEL	
CODE: STX005486 DESCRIPTION: Linea ThermaCare_leaflet_350x420mm DIE CUT/SUPPLIER: - SIZE: 350x420 mm P 175x70 mm	VERSION N°: 00 DATE: 13/11/2024 UNIT OF MEASUREMENT: mm REDUCTION SCALE: 1:1
NOTES:	

ARTWORK CREATION	READABILITY REQUIREMENT	COLOR LIST (not mandatory)	HOLDER APPROVAL SIGNATURE
PRODUCT NAME: LEAFLET TC KNEE 8HR 2CT BE FR CH IE DESIGN CODE: PSX007059 VERSION N°: 01 BRAILLE REFERENT: NA BRAILLE TEXT: NA	MINIMUM FONT SIZE pt: 8,5 SPACE BETWEEN LINE pt: 10,2 PICTOGRAMS: NA (compliance/no compliance) FRONT	1. 186 2. K	</

This design is property of Angetini Pharma s.p.a and may not be reproduced or modified in whole or in part without the written permission of Angetini Pharma s.p.a.

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01029620

STANDARD INFO PANEL	
CODE: STX005484	VERSION N°: 00
DESCRIPTION: Linea ThermoCare_leaflet_350x420mm	DATE: 13/11/2024
DIE CUT/SUPPLIER: -	UNIT OF MEASUREMENT: mm
SIZE: 350x420 mm	REDUCTION SCALE: 1:1
P 175x70 mm	
NOTES:	

ARTWORK CREATION		READABILITY REQUIREMENT		COLOR LIST (not mandatory)	HOLDER APPROVAL SIGNATURE
PRODUCT NAME: LEAFLET TC KNEE 8HR 2CT BE FR CH IE DESIGN CODE: PSX007059 VERSION N°: 01 BRAILLE REFERENT: NA BRAILLE TEXT: NA		MINIMUM FONT SIZE pt: 8,5 SPACE BETWEEN LINE pt: 10,2 PICTOGRAMS: NA (compliance/no compliance) BACK		1. 186 2. K	
NOTES:					
This design is property of Angeline Pharma S.p.A and may not be delivered or admitted to another party in original or in copy. (for client out of DAN System only)					

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Angeline Pharma S.p.A., Viale Amelia 70 - 00181 Rome (Italy)